Cervitec® Liquid

EN Instructions for Use

– Alcohol-free mouth rinse

DE Gebrauchsinformation

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– Bain de bouche, sans alcool

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– Enjuague bucal libre de alcohol

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SV Bruksanvisning

– Alkoholfri munsköljning

DA Brugsanvisning

– Alkoholfrit mundskyllemiddel

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NO Bruksanvisning

– Alkoholfri munnskyllevæske

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– Alkoholvrije mondspoeling

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English

Alcohol-free mouth rinse

Description: Cervitec Liquid is a ready-to-use mouth rinse without alcohol with 0,1% chlorhexidine digluconate, xylitol and D-panthenol. The components inhibit the growth of bacteria that may cause inflammation in the oral cavity, periodontal diseases, caries or bad breath.

Recommended application: For an intensive, temporary mouth rinsing regimen; to maintain the healthy condition of the oral mucosa, the gums and the dental supporting tissue; before and after implant, periodontal, dental or orthodontic surgical procedures; in case of high caries risk; in case of restricted oral hygiene ability; to counteract bad breath.

Rinse mouth with 10-15 ml Cervitec Liquid (dosing cup) twice a day for 30-60 seconds after tooth brushing – unless otherwise recommended. Do not swallow. Thoroughly rinse off any tooth paste residue before using Cervitec Liquid.

Tip: Do not rinse with water after use! This supports the effect of Cervitec Liquid.

Notes: Do not swallow. Not suitable for children under 6 years of age unless medically recommended. Any discoloration of the teeth after prolonged application can be removed by means of professional tooth cleaning. After application over longer periods of time, slight impairment of the sense of taste may occur, which will disappear once the treatment is discontinued.

Keep out of the reach of children. For use in dentistry only.

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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